# smartseal® cervi

D

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GB

Instructions for use

## Mono-component dentine sealing varnish, light-curing

**smartseal® cervi** for the protection of exposed root dentin. This is achieved with a permanent adhesive protective coating and sealing of the dentine tubules. This results in a significant reduction of cervical sensitivity. **smartseal® cervi** strengthens the structure and increases the mechanical stability of root dentine. **smartseal® cervi** possesses high abrasion strength due to a high level of linked multifunctional (meth)acrylates which protect against mechanical abrasion. At the same time, sealing provides protection against the erosive effect of acids.

Fig. 1

Fig. 2

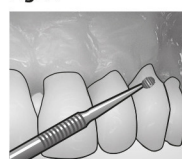
Fig. 3

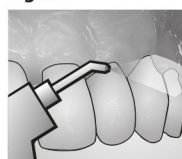
Fig. 4

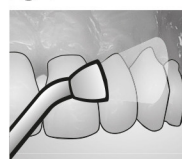
Fig. 5

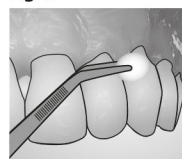
Fig. 6

### Application

Clean the dentine surface concerned professionally. The surface-dry the area carefully with an oil- and water-free air jet. The surface must remain moist, but should not shine wet. Avoid excess drying of the dentine under all circumstances. Place a small amount of **smartseal® cervi**, for max. 3 areas, in a disposable dish or similar (Fig. 1), pick up with the applicator (Fig. 2) and apply to the dentine surface in a single step (Fig. 3). A distinct film should be visible. After an exposure time of 20 sec., overblow with gentle air flow (Fig. 4) to remove volatile components. Polymerise **smartseal® cervi** for 20 sec., at a distance of approx. 2 mm from the lamp (Fig. 5). Expose larger areas several times. Wipe the inhibition coating with an alcohol-soaked cotton pellet or swab (Fig. 6). Then check sulcus for possible excess or residues and remove these with a suitable instrument. After completion, rinse the oral cavity thoroughly.

### Important notes on processing

- Close bottle tightly immediately after use (volatile components).
- Store bottle in upright position.
- Remove excess material from the bottle after each application.
- Only expose briefly to dentist lights as this initiates polymerisation and increases viscosity.
- Avoid contact with mucous membranes, eyes and skin. In case of accidental contact, rinse with adequate running water, consulting an ophthalmologist if necessary.
- A separation of working areas is indicated, i.e. with a coffer dam.
- Non-cured material can lead to irritation and sensitisation upon contact with skin or mucous membranes.
- Do not use in combination with other materials or their residues, such as styptic solutions, eugenol, hydrogen peroxide, or sodium hypochlorite.
- Do not use **smartseal® cervi** if subgingival re-adhesion is expected.
- Do not use near the pulp or the root canal area.
- Inspection and, if necessary, renewal of the sealing coating is recommended at intervals.

### Note

Adverse events are not be expected if used as instructed. Immune reactions such as allergies or irritations can, however, not be excluded in principle. If in doubt, it is recommended to perform an allergy test prior to use.

Only to be used by trained specialist personnel.

DETAX is not liable for any damage caused by improper application.

### Areas of application:

- Stabilisation and protection against erosion, abrasion and abfraction of exposed root dentine
- Reducing sensitivity in cervical areas

### Contra-indication:

**smartseal® cervi** may not be used in case of known allergies to one or more of the ingredients.

### Technical specifications:

- **Volume:** 5 ml bottle
- **Dosage and application:** Add liquid, apply directly with application tip
- **Product colour:** light yellow, transparent
- **Curing:** Halogen lamp > 500 mW/cm<sup>2</sup> (approx. 400-500 nm): approx. 20 sec. LED > 300 mW/cm<sup>2</sup>: approx. 20 sec. Distance 2 mm
- **Processing:** At room temperature
- **Storage:** Store in upright position in a dry and cool place, protect from light and heat

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